Test Title	Cyanide
Sample	1. Specimen type: Blood
Requirements	
Sampling	1. Sample container: Vacutainer blood collection tube (4 mL size, lavender
Materials	top- EDTA anticoagulant)
Procedural	 Chemical Threat Test Request Form for sample submissions
Notes	2. For all samples suspected of containing a chemical threat agent contact
	the IBL Emergency Preparedness Section before sample collection
	and/or submission. TEL: 208-334-2235
	3. Submit a blank vacutainer tube along with the sample tube.
Shipping	Temperature/preservative instructions:
Instructions	a. Samples should be stored and transported at refrigerator
	temperatures (2-8°C).
	2. Packaging:
	a. Package sample in a leak-proof container for transport as per
	DOT or other shipping regulations such as IATA.
	b. Ship as a Category B biological substance.
	3. Ship to:
	Idaho Bureau of Laboratories
	ATTN: LRN-C Laboratory
	2220 Old Penitentiary Rd
	Boise, ID 83712
Reporting and	1. TAT: 1-5 business days
Turnaround	
Time (TAT)	