

SUBMITTING TB SPUTUM SAMPLES TO STATE LAB

Note: Do not open inner plastic container following sample collection.

Step 1

Use State Lab TB kit. Visit www.statelab.idaho.gov to request kits.



Step 5

Place the plastic container into the white container. Tightly close the white container with the cap.



Step 2

Tightly close the labeled plastic tube with the blue cap. Seal the cap with tape provided. Wrap the tube in the absorbent material provided.



Step 6

Place the white container into a refrigerator. Store until sent to State Lab (Attn: TB Lab).



Step 3

Place the plastic tube into the plastic container. Tightly close the plastic container with the cap.



Step 7 (Courier)

If sending via courier, package is ready.




Step 4

Wrap completed TB submittal card around plastic container.



Step 7 (FedEx)

If shipping via FedEx, place white container in appropriately labeled shipping container.

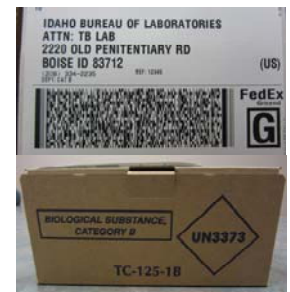

Idaho Bureau of Laboratories
TB Laboratory
2220 Old Penitentiary Rd
Boise, ID 83712
208-334-2235 ext. 253
www.statelab.idaho.gov



IDAHO DEPARTMENT OF
HEALTH & WELFARE