

Adding the Idaho Immunization Program (IIP) Group to Access Zones

The IIP requires all enrolled providers to use a certified, calibrated, continuous recording temperature monitoring device in each vaccine storage unit housing IIP-supplied vaccine. If a provider chooses to utilize IIP-owned PharmaWatch™ equipment, then the provider must grant the IIP access to the PharmaWatch™ portal to view temperature data in all zones obtained by these devices.

The IIP will access the organization's PharmaWatch™ portal to assist with guidance and resolution of reported temperature incidents, to review monthly temperature data, and to assess temperatures before or during an IIP site visit. The IIP will not be responsible for monitoring vaccine storage unit temperatures, or any loss of vaccine.

To grant the IIP access to view the organization's PharmaWatch™ portal, please follow the steps below. Please note that only users with Administrator access may grant IIP access to the portal.

1. Login to the PharmaWatch™ portal.
2. Click the down arrow next to the *People and Places* menu in the dark blue bar near the top of the page.

ID	Location	Short Name	Zone Type	Reporting Sensor	Zone Active	Collection Alerts	Data Alerts	Sensor Assigned	Act
		VFC Frz	Freezer	SM(00:38:a6_1)	✓	✓	✓	✓	⚙️
		VFC Ref	Refrigerator	SM(00:34:1e_1)	✓	✓	✓	✓	⚙️
		VFC Frz	Freezer	SM(00:36:6d_1)	✓	✓	✓	✓	⚙️
		VFC Ref	Refrigerator	SM(00:21:39_1)	✓	✓	✓	✓	⚙️
		VFC Frz	Freezer	SM(00:26:fe_1)	✓	✓	✓	✓	⚙️
		VFC Ref	Refrigerator	SM(00:21:67_1)	✓	✓	✓	✓	⚙️

Adding the Idaho Immunization Program (IIP) Group to Access Zones

- Click *Zones* in the drop-down selection to be redirected to the Zones page of the portal that displays the refrigerator(s) and freezer(s) at the facility.

The screenshot shows the PharmaWatch portal interface. At the top, there is a navigation bar with menu items: Reports, Alerts, People and Places, Setup Hardware, Help, and Select Account. A dropdown menu is open under 'People and Places', listing: Users, Alert Contacts, Accounts, Locations, Zones, and Search. An orange callout box labeled 'Select Zones' points to the 'Zones' option. Below the dropdown, the 'Active Zones' section is visible, containing a table with columns: ID, Location, Short Name, Zone Type, Reporting Sensor, Zone Active, Collection Alerts, Data Alerts, Sensor Assigned, and Action. The table lists six zones with their respective types (Freezer and Refrigerator) and sensor IDs. At the bottom of the table are buttons for 'NEW ZONE', 'VIEW ZONE HISTORY', and 'RELEASE ZONES TO CUSTOMER'.

- Click the pencil icon in the **Action** column on the right side of the screen.

This screenshot shows the same PharmaWatch portal interface as the previous one, but with the 'Action' column in the 'Active Zones' table highlighted. An orange callout box labeled 'Select the Edit Pencil' points to the pencil icon in the 'Action' column of the first row. The table structure and data are identical to the previous screenshot. The footer of the page includes links for Home, American Pharma, Terms, Privacy Policy, and Contact Us, along with the text 'PharmaWatch © powered by American Pharma Technologies Inc. ©2011-2017'.

Adding the Idaho Immunization Program (IIP) Group to Access Zones

5. Check the box next to Idaho Immunization Program in the INCLUDE ZONE IN FOLLOWING GROUPS section.

PharmaWatch™

Change Password Log Out

Reports Alerts People and Places Setup Hardware Help Select Account

Edit Zone >

PharmaWatch ID:

View All Zones

ACTIVE

LOCATION:

ZONE TYPE: Freezer

SHORT NAME:

VFC Frz

Short name is used for menu items and identification - it must be unique at the location

NAME:

VFC Frz

The full name of this zone

DESCRIPTION:

DISPLAY SENSOR TYPE

FTMP

Display Sensor Type is the type of the sensor data used for the Dashboard and other Charts

INCLUDE ZONE IN FOLLOWING GROUPS

Idaho Immunization Program

CALIBRATION

Sensor	Serial # / ID	Cal Due Date
Probe 1:	SN000278	08/22/2017

ACTIVE EQUIPMENT PROFILE

Manufacturer/Model	In Service	Action
No Equipment Profiles are currently active.		

ZONE PURPOSES

Purpose	Action
No Purposes have been assigned.	

6. In the *Allow Group Access?* pop-up box, check the *Allow Access for Idaho Immunization Program* box then click the **ALLOW ACCESS** button.

PharmaWatch™

IDH - Southwest

Dean P. Change Password My Settings Log Out

Reports Alerts People and Places Help

Edit Zone > IDH - Southwest > Emmett > VFC Ref

PharmaWatch ID: 1867

View All Zones

ACTIVE

LOCATION: IDH - Southwest > Emmett

ZONE TYPE: Refrigerator

SHORT NAME:

VFC Ref

Short name is used for menu items and identification - it must be unique at the location

NAME:

VFC Ref

The full name of this zone

DESCRIPTION:

DASHBOARD CHART RANGE

Target Caution Alert

View All Zones

ALLOW GROUP ACCESS?

Allow Access for Idaho Immunization Program

By checking 'Allow Access...', you are agreeing to allow auditors associated with Idaho Immunization Program to access this zone including all graphs, charts, reports, and records for audit/view purposes only.

The auditors will only be able to view the zone. They will not be able to view or edit users or alert contacts, change any settings, or acknowledge alerts.

Check "Allow Access..."

Press "Allow Access"

Home | American Pharma | Terms | Privacy Policy | Contact Us
Pharmawatch is powered by American Pharma Technologies Inc. ©2011-2015

Adding the Idaho Immunization Program (IIP) Group to Access Zones

7. Repeat the steps above for each zone to allow the IIP view-only access to all monitored units at the facility.

For technical assistance, please contact PharmaWatch Tech Support.

Email: support@ameri-pharma.com

Phone: 888-234-5157