



IDAHO DEPARTMENT OF HEALTH & WELFARE  
**DIVISION OF PUBLIC HEALTH**

<b>Test Title</b>	<b>Zika Virus, IgM</b> <b>Methodology: EIA</b>
<b>Sample Requirements</b>	<ol style="list-style-type: none"><li>1. Sample type: Serum only</li><li>2. Separated serum should remain at 20-25°C for no longer than 8 hours; otherwise, serum should be refrigerated at 2-8°C. If testing is not completed within 48 hours of collection, serum should be frozen at or below -20°C.</li><li>3. Rejection criteria: hyper-lipemic, heat inactivated, hemolyzed, icteric or bacterial contamination of serum; avoid multiple freeze/thaw cycles</li><li>4. <a href="#">CDC test criteria</a></li></ol>
<b>Sampling Materials</b>	<ol style="list-style-type: none"><li>1. Sample container: Blood tube, screw cap vial</li></ol>
<b>Procedural Notes</b>	<ol style="list-style-type: none"><li>1. <a href="#">CDC Form 50-34</a></li><li>2. CPT Code: 86790 (IgM)</li></ol>
<b>Shipping Instructions</b>	<ol style="list-style-type: none"><li>1. Temperature/preservative instructions: refrigerated (2-8°C)</li><li>2. Package according to Biological Substance, Category B, shipping guidelines.</li><li>3. Ship to: Idaho Bureau of Laboratories <b>ATTENTION: Virology Laboratory</b> 2220 Old Penitentiary Rd Boise, ID 83712</li></ol>
<b>Reporting and Turnaround Time (TAT)</b>	<ol style="list-style-type: none"><li>1. TAT: 1-3 working days of specimen receipt</li><li>2. Reference range: N/A</li></ol>